

Introducing


DRY-BRIK® II

If you've used Dry & Store previously, you may notice that the enclosed Dry Brik looks a little different from what you're used to. That's because it's the new and improved "Dry Brik II".

Leave the foil label on each Dry Brik intact until you're ready to use it. Store your unused Dry Briks in a cool, dry place for up to a year.

Instructions for use

- 1. Activate the Dry-Brik®:** Use the pull tab at the lower left corner to remove the foil cover. **(Figure 1)**
- 2. Activate the Timestrip®:** Start the timer by firmly pressing your thumb down on the bubble in the middle of the Timestrip. **(Figure 2)** A thin red line will appear in a few minutes, but if not, try again. Do NOT use any sharp objects to press the bubble.
- 3. Place the Dry-Brik in the drying compartment** of your Dry & Store or Zephyr appliance.
- 4. Replace the depleted Dry-Brik after 2 months.**
Each day the red line on the timer window will get a little longer, and at the end of two months it will be completely red. **(Figure 3)** Replace it with a new Dry Brik at that time, and dispose of the old Dry Brik in household refuse.

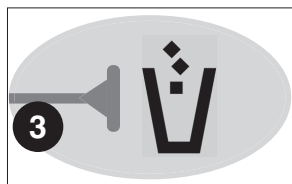

Ear Technology Corporation

Helping people hear better, every day.sm

www.dryandstore.com